



# AMO<sup>®</sup> ENDOSOL

Balanced Salt Solution For Ophthalmic Irrigation (REF 15020)

## New

### Hanger

Integrated in bottle label



### Stopper

Latex free rubber

### Labeling

In 20 languages



- Convenient hanger, integrated in bottle label
- Sterile, isotonic and preservative-free solution
- Convenient 500 ml glass presentation

# AMO<sup>®</sup> ENDOSOL

Balanced Salt Solution For Ophthalmic Irrigation (REF 15020)

## Composition

1 ml of solution contains

Calcium chloride x 2H <sub>2</sub> O	CaCl <sub>2</sub> .2H <sub>2</sub> O	0.48 mg
Potassium chloride	KCl	0.75 mg
Magnesium chloride x 6H <sub>2</sub> O	MgCl <sub>2</sub> .6H <sub>2</sub> O	0.30 mg
Sodium acetate x 3H <sub>2</sub> O	Na-Acet.3H <sub>2</sub> O	3.90 mg
Sodium Citrate x 2H <sub>2</sub> O	Na-Citr.2H <sub>2</sub> O	1.70 mg
Sodium chloride	NaCl	6.40 mg

Citric acid to adjust pH

Water for injection

## Range of uses

For intraocular and topical irrigation of the eye during surgical procedures and for diagnostic and therapeutic measures where irrigation or retention of moisture is required e.g.

- longer irrigation of the anterior and posterior segment e.g. for cataract surgery and vitreoretinal surgery
- surgery on the cornea
- rehydration of the cornea during cornea transplantation
- cooling of the cornea during photorefractive keratectomy
- retention of the anterior segment during trabeculectomy
- reconstruction of the anterior segment during perfusive keratoplasty
- enzymatic zonulolysis
- cauterization

## Instruction for use

An infusion device with air inlet drip chamber should be used as the bottle does not have an additional airway (follow the instructions for the particular infusion device used). This is applied to the container as follows (aseptic conditions):

1. Remove the cap and disinfect the rubber stopper with an alcohol swab.
2. Insert the spike of the infusion device through the target area on the rubber stopper.

The fluid should be flowing before use to remove air from the tube.



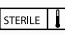



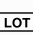



## Storage

- Do not store above 25°C.
- Do not use after the expiry date.

## Packaging:

500 ml glass bottle / 10 bottles per carton

## Symbols used on the package:

	See instructions for use
	Do not reuse
	Steam sterilization
	Do not re-sterilize
	Do not use if package damaged
	Part/Catalogue number
	Batch number
	Use by: (year - month)
	Do not store above 25°C
	Manufacturer

CE 0344

AMO is a trademark owned by or licensed to Abbott Laboratories, its subsidiaries or affiliates.

©2009 Abbott Medical Optics Inc., Santa Ana, CA 92705 www.AbbottMedicalOptics.com

AMO Germany GmbH · Rudolf-Plank-Str. 31 · 76275 Ettlingen · Germany  
www.AbbottMedicalOptics.com

 **Abbott**  
Medical Optics